

# Real-T Endo

## ModuPRO<sup>®</sup> Artificial Endo Module AE13-15 and Placement of Real-T Endo<sup>™</sup> #14 in Breakaway Insert

Real-T Endo tooth #14 in Breakaway Insert. (Item # RT\_AE401WI\_14RL)



Artificial Endo Module AE13-15. (Item #RT\_AE13-15)



Insert the Breakaway Insert into the AE13-15 module.




Push the Breakaway Insert until all teeth are at the same occlusal plane.




Bottom view of Artificial Endo module. To remove the Breakaway Insert, push the insert from the bottom of the module.



Bottom view of Breakaway Insert. Separate facial and lingual inserts by twisting.

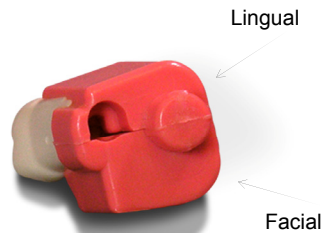

Real-T Endo Tooth #14 is now ready to be viewed.

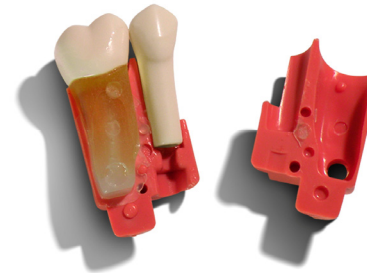

**ACADENTAL**